

AgaMatrix®

WaveSense JAZZ™



Advanced Blood Glucose Monitoring System

OWNER'S GUIDE

Key Features in the AgaMatrix WaveSense JAZZ™

No Coding Required - The AgaMatrix WaveSense JAZZ doesn't require you to enter a code. Insert a test strip and you are ready to test.

Pre/Post Tagging - After you test, tag your reading with the mealtime. Better organize your data and see how your test results change before and after meals.

Positive Feedback - AgaMatrix WaveSense JAZZ will display a smiley face icon if your result is within the target range.

AgaMatrix®

 WaveSense **JAZZ™**

Customer Care: 0800 093 1812

In case of emergency, contact your healthcare professional or emergency medical response.

 0123

Manufacturer:

AgaMatrix, Inc.
7C Raymond Avenue
Salem, NH 03079 USA

©2005-2018 AgaMatrix, Inc.

Meter Firmware ©2004-2018 AgaMatrix, Inc.

All rights reserved. All trademarks and copyrights are the property of their respective owners.

For patent information, see www.agamatrix.com/patents.

8100-02301 Rev J



MDSS GmbH
Schiffgraben 41
30175 Hannover, Germany

Contents

| CHAPTER | TITLE | PAGE |
|-----------|--|------|
| 1 | Getting Started | 4 |
| 2 | Important Health-related Information | 4 |
| 3 | Important Information about Your AgaMatrix WaveSense JAZZ System | 6 |
| 4 | About the AgaMatrix WaveSense JAZZ System | 8 |
| 5 | Meter Basics | 15 |
| 6 | Preparing For a Test With Your AgaMatrix WaveSense JAZZ Meter | 17 |
| 7 | Performing a Control Solution Test | 18 |
| 8 | How To Test Your Blood Glucose Level | 23 |
| 9 | Testing on Your Palm or Forearm | 32 |
| 10 | Data Management Features | 34 |
| 10.1 | <i>Viewing Individual Results</i> | 35 |
| 10.2 | <i>Viewing 14 Day Statistics</i> | 36 |
| 10.3 | <i>Viewing 14, 30, and 90 Day Averages</i> | 36 |
| 10.4 | <i>Viewing Pre/Post Mealtime Averages and Glucose Variability Tracking</i> | 37 |
| 10.5 | <i>Viewing 1 Month, 2 Week, and 3 Day Trend Graphs</i> | 39 |

| CHAPTER | TITLE | PAGE |
|-----------|--|------|
| 11 | Setting the Mealtime Tag | 42 |
| 12 | Setting Meter Volume | 42 |
| 13 | Setting the Time | 43 |
| 14 | Setting the Date | 44 |
| 15 | Setting Hypoglycaemic Warning Alarms | 46 |
| 16 | Setting Hyperglycaemic Warning Alarms | 47 |
| 17 | Setting the Glucose Target Range | 48 |
| 18 | Setting Alarms | 50 |
| 19 | Errors and Troubleshooting | 52 |
| 20 | Display Messages | 57 |
| 21 | Batteries | 60 |
| 22 | Caring For Your Meter and Lancing Device | 63 |
| 23 | Meter Specifications | 64 |
| 24 | Symbols | 65 |

1 Getting Started

Unpack your AgaMatrix WaveSense JAZZ Blood Glucose Monitoring System. Dispose of the packing material properly.

IMPORTANT: To ensure accurate use of meter, verify the time, date and units of measure.

For step-by-step instructions on changing the time and date settings, please refer to chapters 13 and 14.

2 Important Health-related Information

Please Read the Following:

[1] Inaccurate results may occur in severely hypotensive individuals or patients in shock. Inaccurate low results may occur for individuals experiencing a hyperglycaemic-hyperosmolar state, with or without ketosis. Critically ill patients should not be tested with blood glucose meters.

[2] Severe dehydration and excessive water loss may yield inaccurate results. If you believe you are suffering from severe dehydration, consult your healthcare professional immediately.

[3] Patients undergoing oxygen therapy may receive inaccurate results.

[4] Results below 3.9 mmol/L may mean low blood glucose levels (hypoglycaemia).

[5] Results over 13.3 mmol/L may mean high blood glucose levels (hyperglycaemia). Checking ketones may be advisable.

[6] If you get results below 3.9 mmol/L or above 13.3 mmol/L and do not have symptoms of hypoglycaemia or hyperglycaemia, repeat the test. If you have symptoms, or continue to get results below 3.9 mmol/L or above 13.3 mmol/L, follow the treatment plan recommended by your healthcare team or contact your healthcare professional immediately.

[7] If you are experiencing symptoms that are not consistent with your blood glucose test and you have followed all instructions described in the AgaMatrix WaveSense JAZZ Owner's Guide, follow your healthcare professional's recommendations.

[8] If you get repeated error messages and are experiencing symptoms of hypoglycaemia or hyperglycaemia, contact your healthcare professional immediately as this may indicate low or high glucose. If this error code persists on retesting, consult your healthcare professional.

[9] Do not use test strips that are expired or appear to be damaged as they may return inaccurate results.

[10] Healthcare professionals should follow their institution's infection control protocols.

[11] Always follow your healthcare professional's recommendations.

3 Important Information about Your AgaMatrix WaveSense JAZZ System

The AgaMatrix WaveSense JAZZ System should be used:



- [1]** For measuring blood glucose levels from fresh capillary whole blood samples taken from the fingertip, palms (at the base of the thumb), or forearms.
 - [2]** For *in vitro* diagnostic (i.e. outside of the body) use only.
 - [3]** With AgaMatrix WaveSense JAZZ Test Strips and AgaMatrix Control Solution only. Do not use other brands of test strips and control solution with this meter. This may produce inaccurate results.
 - [4]** For use at home (over the counter (OTC)) by persons with diabetes, or in a clinical setting by healthcare professionals, as an aid to monitor the effectiveness of diabetes control.
-

Do not use the AgaMatrix WaveSense JAZZ System for:

- [1]** The diagnosis of diabetes.
 - [2]** Testing the glucose levels of neonates (children under 4 weeks).
 - [3]** Testing glucose levels of arterial or venous blood.
 - [4]** Testing glucose from sites other than the fingertip, palm (at the base of the thumb), or forearm.
-

Possible test sites:

The AgaMatrix WaveSense JAZZ System requires only 0.5 μL of blood to perform a glucose test. For best results, test using blood from the tip of any finger. The palm (at the base of the thumb) and forearm are alternate, but less accurate test sites. Only select soft and fleshy areas to lance. Avoid lancing any bony areas, obvious veins and moles.

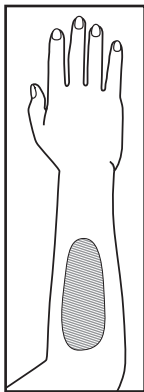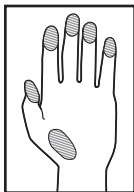

Physiological differences in circulation between the fingertip, palm (at the base of the thumb), and forearm may result in differences in blood glucose measurements between these sites. Differences in glucose concentrations may be observed after eating, taking insulin medication, or exercise. Changes in blood glucose may be detected at the fingertips before the palm (at the base of the thumb) and forearm.

It is recommended that you use a fingertip sample if:

[1] You are testing for hypoglycaemia or if you suffer from hypoglycaemia unawareness,

[2] You are testing your blood glucose within 2 hours of eating, taking insulin or medication, exercising, or

[3] Your blood glucose results obtained from the palm (at the

base of the thumb) or forearm are not consistent with the way you feel.

CAUTION: Please read all the instructions provided in this Owner's Guide and practice the testing procedures before using the AgaMatrix WaveSense JAZZ Blood Glucose Monitoring System. Blood glucose monitoring should be done under the guidance of your healthcare professional.


4 About the AgaMatrix WaveSense JAZZ System

Your AgaMatrix WaveSense JAZZ System includes:

- AgaMatrix WaveSense JAZZ Blood Glucose Meter
- Two Pre-installed CR2032, 3 Volt, Lithium Batteries
- Lancing Device with Cap
- Clear AST Lancing Device Cap
- 30 Sterile Lancets
- Compact Carrying Case
- 1 Vial of 25 Test Strips
- Control Solution
- Owner's Guide
- Quick Start Guide
- Prescription Card
- Registration Card
- Two Replacement CR2032, 3 Volt, Lithium Batteries
- Log Book

About the AgaMatrix WaveSense JAZZ Test Strip

With your AgaMatrix WaveSense JAZZ System, you can test your blood glucose on your fingertips, palms (at the base of the thumb) or forearms. Testing with the AgaMatrix WaveSense JAZZ System requires a small sample size, 0.5 μ L of blood, about the size of a pinhead.

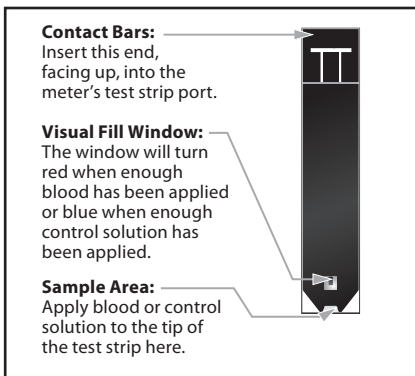

CAUTION: Dispose used test strips properly.

IMPORTANT: Use AgaMatrix WaveSense JAZZ Test Strips only once.

IMPORTANT: Only use AgaMatrix WaveSense JAZZ Test Strips with the AgaMatrix WaveSense JAZZ System.

Important Test Strip Information:

[1] Store the AgaMatrix WaveSense JAZZ Test Strip vial in a cool, dry place at 8 °C to 30 °C (46 °F to 86 °F).

[2] Use AgaMatrix WaveSense JAZZ Test Strips only within the system operating temperature range of 10 °C to 40 °C (50 °F to 104 °F).

[3] Keep away from direct sunlight and heat.

[4] Store your AgaMatrix WaveSense JAZZ Test Strips in their original vial only; never store them in another vial, any other container, or outside the vial.

[5] After removing an AgaMatrix WaveSense JAZZ Test Strip from the vial, immediately close the vial cap tightly.


[6] With clean, dry hands you may gently handle the AgaMatrix WaveSense JAZZ Test Strip when removing it from the vial or inserting it into the meter.

[7] Do not use AgaMatrix WaveSense JAZZ Test Strips beyond the expiration date or 180 days after first opening the vial. This may cause inaccurate results. Write the discard date (180 days from the first opening) on the test strip vial.

[8] Do not bend, cut or alter AgaMatrix WaveSense JAZZ Test Strips.

[9] Apply only fresh capillary blood or control solution to the sample area of the AgaMatrix WaveSense JAZZ Test Strip.

About the AgaMatrix WaveSense JAZZ Meter

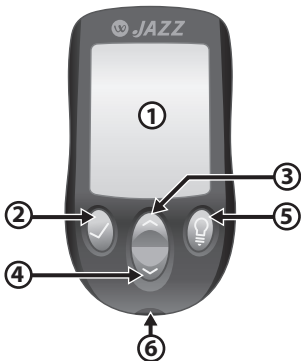

(1) Display: Glucose test results, symbols, messages, and graphs appear here.

(2) TICK (✓) Button: Use the TICK button to confirm an option or make a selection in a menu.


(3) UP Arrow (▲) Button: Use the UP arrow button to scroll backward through options.

(4) DOWN Arrow (▼) Button: Use the DOWN arrow button to scroll forward through options.

(5) BACKLIGHT (💡) Button: Use the BACKLIGHT button to turn the backlight ON and OFF.

(6) Test Strip Port: Insert the AgaMatrix WaveSense JAZZ Test Strip, with the contact bars facing up, into the test strip port.

About the AgaMatrix WaveSense JAZZ Meter Display Symbols/ Messages

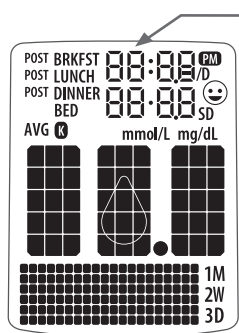

(1) Date and Time:

Appears here.

12:30
04:05

(2) Glucose Test Results:



(3) Low Battery Message:

Appears when battery #2 in your meter is running low.



(4) Dead Battery Message: Appears when battery #2 in your meter is dead.



(5) Temperature Symbol: Appears when the meter is outside its operating temperature range of 10 °C to 40 °C (50 °F to 104 °F).

TEMP

(6) Ketone Symbol: Appears when your glucose test results are higher than 13.3 mmol/L.

K

(7) Control Solution message: Appears after you have performed a control solution test.

Ct r1

(8) Units of Measurement symbol: Results will be displayed in mmol/L.

mmol/L

(9) Apply Sample to Test Strip message: The flashing droplet symbol and the word “fill” will signal you to apply a sample to the test strip.



(10) Volume Message: Can be set to ON or OFF.




(11) 1 Month, 2 Week, 3 Day Indicators: The appropriate number of days will appear when viewing graphical averages.

1M
2W
3D

(12) 1,2,3 Countdown: The 1, 2 and 3 symbols will appear during a glucose test.

1
2
3

(13) Graphing Area: Areas of the dot matrix will appear during a test and when displaying graphs.

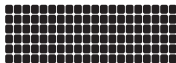

(14) Happy Face Symbol: The happy face is displayed with glucose test results that fall within specific pre and post meal target ranges.



(15) Mealtime Tags: The mealtime tags appear after a test is complete and will help categorize your results.

POST BRKFST
POST LUNCH
POST DINNER
BED

(16) /Day, SD, and AVG Symbols: These symbols appear in the meter's data management functions.

/D
SD
AVG

About the AgaMatrix Lancing System

(1) Cocking Handle: Cocks the device so it is ready to lance.

(2) Depth Selector: Adjusts how deep the lancet will lance the skin.

(3) Depth: Select from 5 levels of depth for skin penetration, indicated by a large dot (highest setting) to the small dot (lowest setting).

(4) Release Button: Fires the lancet.

(5) Lancet: Lances the skin to produce a drop of blood.


(6) Lancet Cover: Covers the lancet for safety.

(7) Lancing Device Cap: Covers the lancet when in lancing device.

(8) Alternate Site Testing Cap: An AST clear lancing device cap used for obtaining a blood sample on the palm (at the base of the thumb) or forearm.

(9) Lancet Ejector: Safely ejects the used lancet.


(10) Lancet Holder: Holds the lancet into place inside the lancing device.

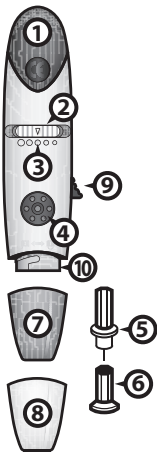

METER SYSTEM CHECK

Each time your meter is activated, the meter will perform a meter system check. All display segments and the backlight will briefly flash ON.

IMPORTANT: *If all the display segments DO NOT flash ON (each time your meter is activated), do not use the meter. Call Customer Care immediately.*

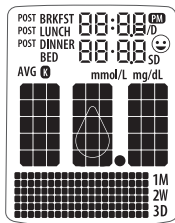**HOW TO ACTIVATE THE METER AND ENTER MENU MODE**

[1] When your meter is deactivated, press the TICK (✔) button to enter the menu mode.

[2] Press the DOWN (▼) or UP (▲) arrow buttons to view the options: MEM, STATS, AVG, MEAL, GRAPH, TIME, DATE, TAG, RANGE, BEEP, HYPO, HYPER, and ALARM. To view/alter any of these options or settings press the TICK (✔) button.

HOW TO ACTIVATE THE METER AND VIEW INDIVIDUAL RESULTS

[1] When your meter is deactivated, press the TICK (✔) button to enter menu mode. The word “MEM” is displayed. Press the

TICK (✔) button again to see your most recent test result.

[2] Press the DOWN (▼) or UP (▲) arrow buttons repeatedly to scroll thru every test result, starting from the most recent and ending with the oldest.

HOW TO ACTIVATE THE METER AND VIEW MEALTIME AVERAGES

[1] When your meter is deactivated, press the TICK (✔) button once to enter Menu mode. The word "MEM" is displayed. Press the DOWN (▼) or UP (▲) arrow buttons until the word "MEAL" appears in the lower part of the display. Press the TICK (✔) button again to see your mealtime averages.

[2] Press the DOWN (▼) or UP (▲) arrow buttons repeatedly to scroll through your Breakfast, Lunch, Dinner, and Bedtime Averages.

HOW TO DEACTIVATE THE METER

To deactivate the meter, press and hold the TICK (✔) for 2 seconds. If the meter is inactive for 3 minutes, the meter will be deactivated automatically.


HOW TO ACCELERATE THROUGH OPTIONS

Hold down the DOWN (▼) or UP (▲) arrow buttons to accelerate through options or results.

6 Preparing For a Test With Your AgaMatrix WaveSense JAZZ Meter

[1] Insert an AgaMatrix WaveSense JAZZ Test Strip: Insert a new AgaMatrix WaveSense JAZZ Test Strip into the meter. Make sure you insert the test strip, with the contact bars facing up, into the test strip port. The meter will be activated automatically.

Note: If you do not start the test within 5 minutes, the meter will be deactivated. To reactivate your meter, press the TICK (✓) button or remove the unused AgaMatrix WaveSense JAZZ Test Strip and reinsert it into the meter.

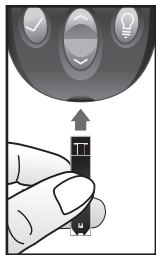

[2] Meter Ready for Test: The meter now displays the blood drop symbol and the word "Fill". The AgaMatrix WaveSense JAZZ System is now ready for you to apply control solution or blood.

The AgaMatrix WaveSense JAZZ System has 2 different levels of control solution: Normal and High. Select the level of control solution you want to use.

The control solution is used to:

- [1]** Ensure that your AgaMatrix WaveSense JAZZ Meter and Test Strips are working together properly.
 - [2]** Practice testing without having to use your own blood.
-

Control solution tests should be performed when you:

- [1]** First get your meter.
- [2]** Suspect that your meter or test strips are not working properly.
- [3]** Think your test results are not accurate.
- [4]** Have dropped or damaged your meter or exposed your meter to liquids.
- [5]** Are advised by your healthcare professional to do so.

Control solution should only be used for 90 days after first opening the bottle or until the expiration date printed on the label, whichever comes first.

IMPORTANT: Count forward 90 days from the date you open a new bottle of AgaMatrix Control Solution. This is your discard date. Write the discard date on the AgaMatrix Control Solution bottle.

Important AgaMatrix Control Solution information:

[1] Use only AgaMatrix Control Solution with your AgaMatrix WaveSense JAZZ System.

[2] Always replace the cap on the control solution bottle immediately after use.


[3] Do not add water or any other liquid to control solution.

[4] Control solution tests should be performed within the system operating temperature range of 10 °C to 40 °C (50 °F to 104 °F).


How to perform a control solution test:

[1] Inserting an AgaMatrix WaveSense JAZZ Test Strip: Insert a new AgaMatrix WaveSense JAZZ Test Strip into the meter. The meter will be activated when the test strip is fully inserted.

[2] Meter Ready for Test: The meter now displays the droplet symbol and the word "Fill". The AgaMatrix WaveSense JAZZ System is now ready for you to apply control solution. You do not need to set a calibration code for this meter.

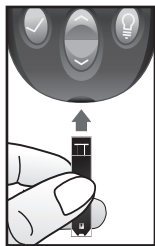


[3] Applying Control Solution: Shake the control solution bottle. Discard the first drop of control solution and wipe the bottle tip clean using a clean tissue or paper towel. Dispense a second drop onto a clean surface, such as an unused resealable plastic bag.

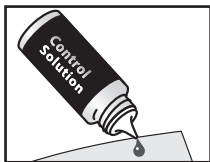


Bring the tip of the test strip to the control solution sample. The AgaMatrix WaveSense JAZZ System will automatically detect if a test is performed with AgaMatrix Control Solution.

This is the visual fill window.
DO NOT PUT CONTROL SOLUTION HERE.

Apply control solution here.

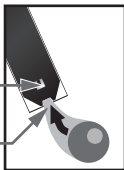


When control solution is applied, the visual fill window on the AgaMatrix WaveSense JAZZ Test Strip turns blue. Your meter will beep once indicating that you should remove the tip of the AgaMatrix WaveSense JAZZ Test Strip from the control solution sample.

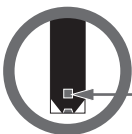

The entire visual fill window will turn blue.

On the meter display, the droplet symbol fills. The scrolling bar and 1-2-3 indicates that the meter is calculating your results.

Droplet Symbol:

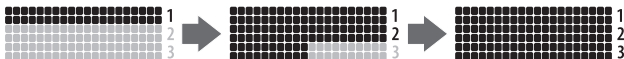

[4] Viewing Control Solution Test Result:

The result will appear on the display and, if the volume is enabled, your meter will beep once. The date and the abbreviation “Ctrl” will also appear on the display.



Compare the result of your control solution test to the range printed on the AgaMatrix WaveSense JAZZ Test Strip vial label for the control solution level (Normal or High) that you used. Your control solution result should fall within this range. The control solution test result is stored in memory.



To deactivate the meter, remove the used AgaMatrix WaveSense JAZZ Test Strip. Dispose of used materials properly.

If your control solution test results are out of the range, repeat the test. If the results from the control solution tests continue to fall outside that range, do not use the AgaMatrix WaveSense JAZZ System to test your blood glucose. Call Customer Care.

Normal: 4.4-6.7 mmol/L

High: 12.2-17.8 mmol/L

FOR EXAMPLE ONLY.

Refer to your test strip vial label for corresponding range.

IMPORTANT: Out-of-range results may be due to one or more of the following factors:

[1] Expired or defective control solution.

[2] Expired or defective test strip.

[3] Error in performing test.

[4] Watered-down control solution.

[5] Meter malfunction.

[6] Control solution test done outside the system operating temperature of 10 °C to 40 °C (50 °F to 104 °F).

[7] Failure to shake the control solution bottle vigorously before using.

[8] Failure to discard the first drop of control solution and to wipe the bottle tip clean.

CAUTION: Results from control solution tests do not reflect your blood glucose level. The control solution range is a target range for control solution only. It is not a target range for your blood glucose level.

8 How to Test Your Blood Glucose Level


CAUTION: To ensure accurate results, wash your hands with warm, soapy water and dry before every test. Make sure there is no grease, oil or lotion on the test site. Use lancets only once. Do not use any lancet or lancing device that has been used by another person. Unwashed hands may lead to inaccurate results.

How to test your blood glucose level:

[1] Preparing the Lancing Device: Use the gray cap for testing on a fingertip and the clear cap for testing on the palm (at the base of the thumb) or forearm.

Remove Lancing Device Cap:


Twist the cap counter-clockwise while holding the body of the lancing device, to remove the cap.

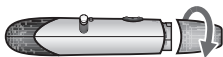

Insert New Lancet: Insert a new lancet firmly into the lancet holder until it comes to a stop.



To expose more of the lancet and enable you to remove the lancet cover easily, press the release button to fire the lancing device.

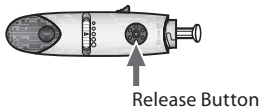

Twist the Lancet Cover Off: Hold the lancet firmly in place with one hand and use your other hand to twist off the lancet cover. Twist/pull in a straight direction while removing the cover to avoid bending the lancet tip.




Note: Do not touch the lancet tip, to retain sterility.

Replace Lancing Device Cap: Replace the lancing device cap by twisting it clockwise onto the lancing device body. Ensure lancing device cap is securely locked in place; otherwise the device may not function as intended.



Set the Lancing Level: The lancing device offers 5 depth settings. Select the skin penetration depth by sliding the depth selector to the desired depth.

The 5 dots represent the 5 levels of depth settings, with the largest dot (highest setting) on the left progressing to the smallest dot (lowest setting) on the right. If you have never lanced before, it is recommended that you start at the third dot. If necessary, set the depth level to a deeper setting to produce enough blood to fill the test strip (0.5 μ L).

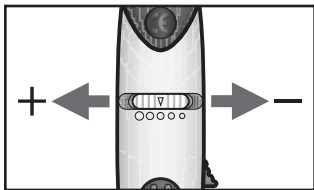


Cock the Handle: Pull the cocking handle out until it clicks. The lancing device is now ready for use.



[2] Insert an AgaMatrix WaveSense

JAZZ Test Strip: Insert a new AgaMatrix WaveSense JAZZ Test Strip into the strip port of the meter. Make sure you insert the test strip, with the contact bars facing up, into the test strip port. The meter will be activated when the test strip is fully inserted.

Note: If you do not start the test within 5 minutes, the meter will be deactivated. To reactivate your meter, take out the unused AgaMatrix WaveSense JAZZ Test Strip and reinsert it into the meter.

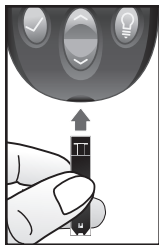


Meter Ready for Test: The meter now displays the droplet symbol and the word “Fill”. The AgaMatrix WaveSense JAZZ System is now ready for you to apply blood. You do not need to set a calibration code for this meter.

[3] Lancing: To ensure accurate results, wash hands with warm, soapy water and dry before every glucose test. For details on possible test sites, refer to chapter 3.

For instructions on how to test on your palm or forearm, please refer to chapter 9.


Note: Keep your hand warm or gently massage the site you are lancing to stimulate blood flow.

Lance Test Site: Press the lancing device against the site to be lanced. Press the release button.

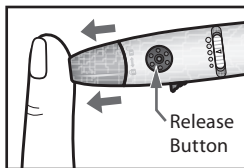

Express Blood Drop: When testing on a finger tip, wipe away the first blood drop that appears using a clean tissue or paper towel. Squeeze from the base of your finger up towards the tip, until a second small blood drop forms.

Do not squeeze directly around the lanced area.

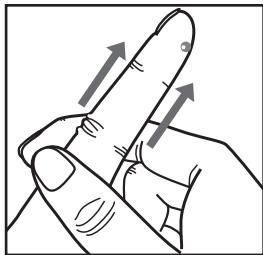


If necessary, set the lancing device to a deeper setting or use a larger lancet to produce enough blood.


[4] Applying Blood Sample

Bring Test Strip to Blood Sample: The meter now displays the droplet symbol. This means your AgaMatrix WaveSense JAZZ System is now ready for you to apply blood. Immediately bring the AgaMatrix WaveSense JAZZ Test Strip to the blood sample.

The test strip fills from the tip. Do not apply blood on the top surface of the test strip.

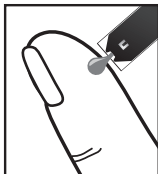

The AgaMatrix WaveSense JAZZ Test Strip acts like a sponge and draws the blood into the test strip through the sample area. The visual fill window of the AgaMatrix WaveSense JAZZ Test Strip will turn red.

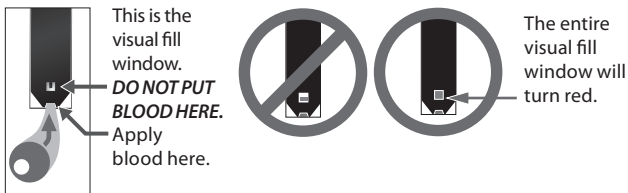

WARNING: If the visual fill window does not completely fill, you may get an inaccurate result. Retest and ensure that the visual fill window is completely full.

IMPORTANT:

[1] Remove the test strip from the blood sample as soon as you hear the beep.

[2] Do not press the AgaMatrix WaveSense JAZZ Test Strip against the test site.

[3] Do not scrape blood onto the AgaMatrix WaveSense JAZZ Test Strip.

[4] Do not apply blood to the top side of the AgaMatrix WaveSense JAZZ Test Strip.

[5] Do not apply blood to the AgaMatrix WaveSense JAZZ Test Strip when the test strip is out of the meter.

[6] Do not put blood or foreign objects into the AgaMatrix WaveSense JAZZ test strip port. Doing so may lead to system failure.


[7] Do not apply more blood after beep.

[8] Carefully read the test results on the display before making any treatment decisions.

Remove Test Strip from Blood Sample when you hear the beep:

Your meter will beep once, when you should remove the tip of the AgaMatrix WaveSense JAZZ Test Strip from the blood sample.

On the meter display, the droplet symbol fills. The scrolling bar and 1-2-3 indicates that the meter is calculating your results.

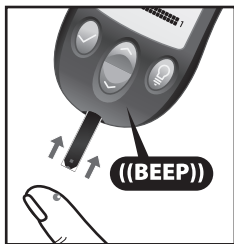

Droplet Symbol: 



Your test is completed when your blood glucose test result with time and date is shown on the display. If the volume is ON, you will hear a beep. Your test result is now stored in the memory.



[5] Test Result Mealtime Tagging and Feedback

If the Mealtime Tag Feature is Enabled: The AgaMatrixWaveSense JAZZ Meter allows you to add a tag to your test result. Tagging results allows you to view averages before and after meals to see what effect food has on your blood glucose levels.

After your result appears, a tag will flash on the display suggesting a mealtime tag (Breakfast, Post Breakfast, Lunch, Post Lunch, Dinner, Post Dinner, and Night). For example, if you test at 12:30, the meter will flash the word “POST LUNCH.”

If the tag is incorrect, you can change it by pressing the DOWN (⏮) or UP (⏭) arrow buttons. Pressing the UP (⏭) arrow button would change the tag to Lunch, pressing the DOWN (⏮) arrow button would change the tag to Dinner. Pressing the TICK (✓) button confirms the mealtime tag and it stops flashing. Your test result’s mealtime tag is now stored in the memory. You can still change this setting by pressing the DOWN (⏮) or UP (⏭) arrow buttons, if you haven’t removed the test strip from the meter yet.

If you don’t want to tag a reading, scroll through the options by pressing the DOWN (⏮) or UP (⏭) arrow buttons. One of the options will leave the tag field blank. If you do not want to tag meals, this feature can be turned OFF in the menu mode.

Example of
Mealtime Tag:
POST LUNCH



Example of
Mealtime Tag:
POST BREAKFAST



Example of
Mealtime Tag:
NO TAG (BLANK)



Tagging Values:

The AgaMatrix WaveSense JAZZ Meter suggests mealtime tagging values according to this chart:

| |
|------------------------------------|
| Breakfast: 5:00 - 7:59 |
| Post Breakfast: 8:00 - 9:59 |
| Lunch: 10:00 - 11:59 |
| Post Lunch: 12:00 - 14:59 |
| Dinner: 15:00 - 17:59 |
| Post Dinner: 18:00 - 20:59 |
| Night: 21:00 - 4:59 |


Feedback: Depending on the mealtime tag and your test result, the meter will display positive feedback in the form of a happy face. The happy face symbol is displayed with glucose test results that fall within specific pre and post meal target ranges.




[6] Final Steps

Remove the Test Strip: Remove the used AgaMatrix WaveSense JAZZ Test Strip. The AgaMatrix WaveSense JAZZ Meter will automatically deactivate.


IMPORTANT: If you do not remove the used test strip after 3 minutes of testing, the meter will deactivate. To activate your meter with a used test strip still in the meter, press the TICK (🕒) button once. Do not attempt to apply blood or control solution to an already used test strip.

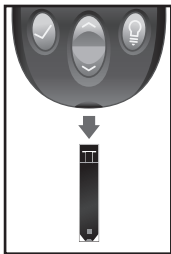

Remove the Lancet: When you have finished testing, twist off the lancing device cap, while holding the body of the lancing device. Be careful not to touch the lancet tip.

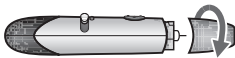

Extract the lancet by pushing the lancet ejector downwards, and ejecting the lancet safely into an appropriate sharps container.

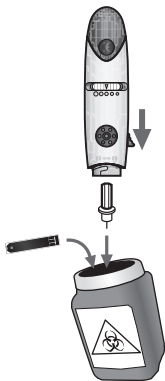

Replace the lancing device cap by twisting the cap clockwise onto the lancing device body.



Discard Properly: Follow your healthcare professional's instructions for disposal.

WARNING: *The lancing device and lancets should only be used by one person. Never share lancing devices or lancets. Used test strips, lancets, and lancing devices may be considered biohazardous or medical waste in your city or town. Follow all local regulations for proper disposal.*

9 Testing on Your Palm or Forearm

Testing glucose levels with blood obtained from the palm (at the base of the thumb) or forearm may reduce the pain of testing. The technique for testing on the palm and forearm is slightly different than testing on the finger. Always discuss changes to your testing habits with your healthcare professional.

IMPORTANT: *When testing on the palm (at the base of the thumb) or forearm, you may need to set the lancing device to a deeper setting or use a larger lancet to obtain enough whole blood to perform a test. Consult your healthcare professional for a recommendation.*

CAUTION: *To ensure accurate results, wash your hands with warm, soapy water and dry before every test. Clean and make sure there is no grease, oil or lotion on the test site.*


Follow Chapter 8, steps 1-2, to begin blood glucose testing.

For testing on your palm or forearm:


[1] Prepare Lancing Device: Always use the AST clear lancing device cap on the lancing device.

[2] Rub Test Site: To increase the blood flow in your palm (at the base of the thumb) or forearm, rub the

Palm:

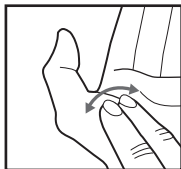


Forearm:

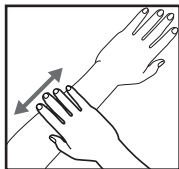



area that you will be testing. Ensure that your palm or forearm is facing up and held level.

[3] Lance Test Site: Press the tip of the clear cap against your skin on the palm of your hand (at the base of the thumb) or forearm. Hold the tip to the test site for a few seconds before pressing the release button.

Palm:

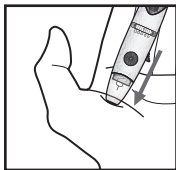


Forearm:

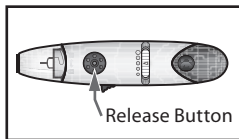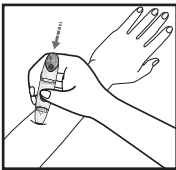


[4] Express Blood Drop: After lancing, hold the lancing device against your skin until a blood droplet forms. Make sure the blood drop is large enough to completely fill the visual fill window of the test strip. Pull the lancing device directly off your palm or forearm to ensure that you do not smear the blood drop.


Palm:

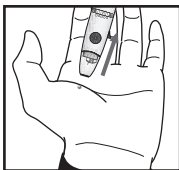


Forearm:

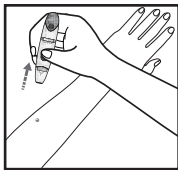

Palm:

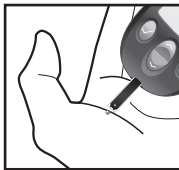

Forearm:

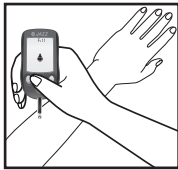

Return to Chapter 8, step 4 to continue with testing instructions.

10 Data Management Features

IMPORTANT: Results, averages and graphs stored in memory should only be used as a reference. Do not make treatment decisions based solely on stored results, averages and graphs. Consult with your healthcare professional regarding treatment options.

This chapter covers the steps on how to view your:

- Individual Results (1865 past results, with time and date)
 - 14 Day Statistics
 - 14, 30, and 90 Day Averages
 - Pre/Post Mealtime Averages and Glucose Variability Tracking
 - 1 Month, 2 Week, 3 Day Trend Graphs
-

How to access all data management features:

All past test results are accessed the same way. The following instructions will explain how to get to the different types of data.

[1] When your meter is deactivated, press the TICK (🗑️) button to enter menu mode. The word “MEM” is displayed.

[2] Press the DOWN (⏴) arrow button to scroll forward and press the UP (⏵) arrow button to move backward through the categories of data.

[3] Press the TICK (🗑️) button to enter into a category and view features.

About averages and standard deviations:

Control solution, out-of-temperature-range results, and results without a valid time or date are not included in averages, standard deviations and trend graphs. Hi and Lo test results will be included in averages as values of 33.3 mmol/L and 1.1 mmol/L respectively. 14, 30, and 90 day averages and trend graphs are calculated by taking the average of all readings within the respective time period. Standard deviations, averages and trend graphs do not include the current day in their calculations.

The standard deviation is an expression of the variation of readings from the average reading. The greater this value the more variation exists between individual readings and the average. The standard deviation feature is a way to track your glucose variability.


10.1 Viewing Individual Results

[1] When your meter is deactivated, press the TICK (✓) button once to enter menu mode. The word "MEM" is displayed.



[2] Press the TICK (✓) button again to enter the individual results section.

[3] Press the DOWN (▼) arrow button to scroll through the individual readings. Press the UP (▲) arrow button to scroll backwards.

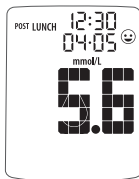

Note: When scrolling through, the oldest test result will cycle back to the most recent test result. To return to the menu, press the TICK (✓) button.

10.2 Viewing 14 Day Statistics

The stats feature allows you to compare your average test result, the standard deviation between test results, and the average number of tests per day over the last 14 day period.

[1] When your meter is deactivated, press the TICK (✔) button once to enter menu mode. The word “MEM” is displayed.

[2] Press the DOWN (▼) or UP (▲) arrow buttons until the word “STATS” appears on the display.


A rectangular display showing the word "STATS" in a pixelated font.

[3] Press the TICK (✔) button again to enter the statistics section.

Average number of tests per day

Standard Deviation

Average Test Result

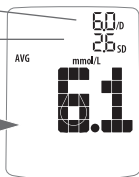

Note: To return to the menu, press the TICK (✔) button.

10.3 Viewing 14, 30, and 90 Day Averages

[1] When your meter is deactivated, press the TICK (✔) button once to enter menu mode. The word “MEM” is displayed.

[2] Press the DOWN (▼) or UP (▲) arrow buttons until the word “AVG” appears on the display.




A rectangular display showing the word "AVG" in a pixelated font.

[3] Press the TICK (✔) button again to enter the 14, 30, and 90 day averages section. The display now shows your 14 day average and average number of tests per day.

[4] Press the DOWN (⬇️) arrow button. The display now shows your 30 day average and average number of tests per day.

[5] Press the DOWN (⬇️) arrow button. The display now shows your 90 day average and average number of tests per day.


Note: Press the UP (⬆️) arrow button to scroll backwards. When scrolling, you will cycle through the 14, 30 and 90 Day Averages. To return to the menu, press the TICK (✔️) button.

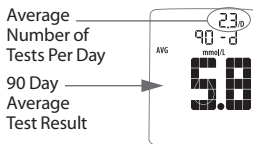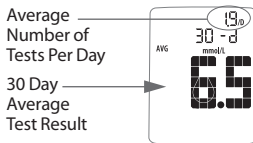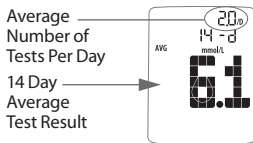

10.4 Viewing Pre/Post Mealtime Averages & Glucose Variability Tracking

The AgaMatrix WaveSense JAZZ Meter categorizes readings as happening before or after meals. These averages allow you to see how your blood glucose levels vary by time of day. The pre/post mealtime averages and standard deviations only include the first instance of a particular mealtime tag (breakfast, post breakfast, lunch, post lunch, etc.) per day over the last 14 day period. The second and subsequent tests marked with the same time tag (on the same day) are not included. The standard deviation feature is a way to track your glucose variability.

Note: Standard deviation is always displayed with a decimal point.

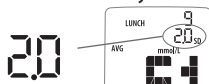

Example of a standard deviation of 2.0

[1] When your meter is deactivated, press the TICK (✓) button once to enter menu mode. The word “MEM” is displayed.

[2] Press the DOWN (⌵) or UP (⌶) arrow buttons until the word “MEAL” appears in the lower part of the display.





[3] Press the TICK (✓) button again to enter the mealtime averages section. The display now shows your breakfast average.



Number of results tagged

Standard Deviation

Average Test Result

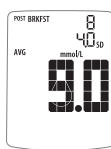
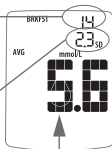


[4] Press the DOWN (⌵) arrow button. The display now shows your post breakfast average.

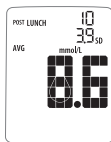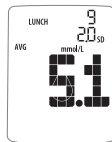


[5] Press the DOWN (⌵) arrow button. The display now shows your lunch average.

[6] Press the DOWN (⌵) arrow button. The display now shows your post lunch average.

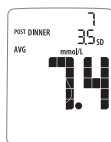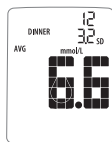

[7] Press the DOWN (⌵) arrow button. The display now shows your dinner average.

[8] Press the DOWN (⌵) arrow button. The display now shows your post dinner average.

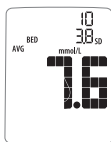

[9] Press the DOWN (⌵) arrow button. The display now shows your bedtime average.

Note: Press the UP (▲) arrow button to scroll backwards. When scrolling, you will cycle through the Pre/Post Mealtime Averages. To return to the menu, press the TICK (✔) button.


10.5 Viewing 1 Month, 2 Week, and 3 Day Trend Graphs

[1] When your meter is deactivated, press the TICK (✔) button once to enter menu mode. The word “MEM” is displayed.


[2] Press the DOWN (▼) or UP (▲) arrow buttons until the abbreviation “GRAPH” appears in the lower part of the display.




[3] Press the TICK (✔) button again to enter the graph section. The display now shows your 1 month graph.

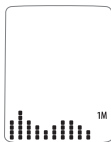

[4] Press the DOWN (▼) arrow button. The display now shows your 2 week trend graph.

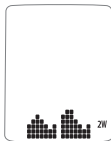


[5] Press the DOWN (▼) arrow button. The display now shows your 3 day trend graph.

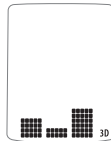


Note: Press the UP (▲) arrow button to scroll backwards. When scrolling, you will cycle through the Trend Graphs. To return to the menu, press the TICK (✔) button.

1 Month Trend Graph:

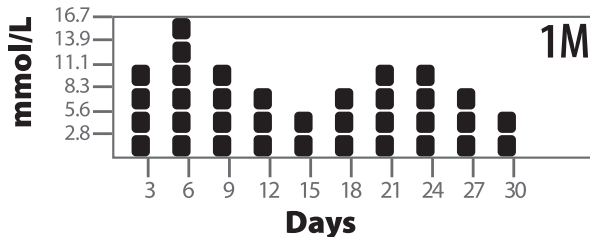


Each bar represents a test result average from 3 days. Each vertical unit represents about 2.8 mmol/L. **The most recent result is on the right side of the screen.**

2 Week Trend Graph:

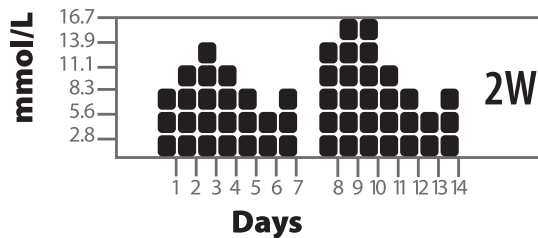


Each bar represents a test result average from 1 day. Each vertical unit represents about 2.8 mmol/L. **The most recent result is on the right side of the screen.**

3 Day Trend Graph:

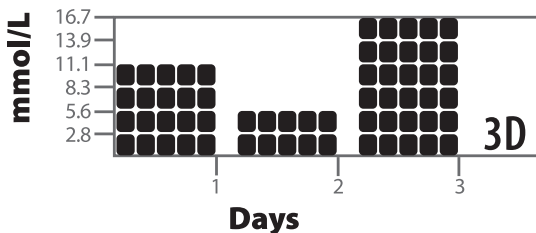

Each bar represents a test result average from 1 day. Each vertical unit represents about 2.8 mmol/L. **The most recent result is on the right side of the screen.**

IMPORTANT: Averages above 16.7 mmol/L will not be represented in the trend graphs.

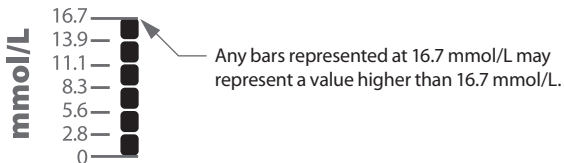

Setting the Mealtime Tag

The AgaMatrix WaveSense JAZZ Meter allows you to turn ON or OFF the feature that tags results as having occurred before or after a meal.


[1] When your meter is deactivated, press the TICK (✔) button once to enter menu mode. The word “MEM” is displayed.

[2] Press the DOWN (▼) or UP (▲) arrow buttons until the word “TAG” is displayed.



[3] Press the TICK (✔) button to enter and edit the tag setting.

[4] Turn the tagging feature ON or OFF with the DOWN (▼) or UP (▲) arrow buttons. Your selection will be highlighted by the blinking of the symbol. Press the TICK (✔) button to confirm your selection and return to the menu.

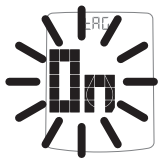

Setting Meter Volume


[1] When your meter is deactivated, press the TICK (✔) button once to enter menu mode. The word “MEM” is displayed.

[2] Press the DOWN (▼) or UP (▲) arrow buttons until the word “BEEP” is displayed.



[3] Press the TICK (✔) button to enter and edit the alarm volume setting.

[4] Turn the volume ON or OFF with the DOWN (▼) or UP (▲) arrow buttons. Your selection will be highlighted by the blinking of the symbol. Press the TICK (✔) button to confirm your selection and return to the menu.

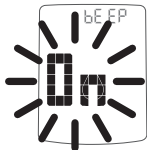

Note: The meter will always produce a beeping sound after you have applied blood or control solution to a test strip. This sound lets you know when to take your blood sample or control solution away from the test strip. You will hear this sound if the volume is set to ON or OFF.

13 Setting the Time

IMPORTANT: Check the Time and Date settings before testing. In order to properly use the meter it is important to ensure that the time and date are set correctly.

[1] When your meter is deactivated, press the TICK (✔) button once to enter menu mode. The word “MEM” is displayed.

[2] Press the DOWN (▼) or UP (▲) arrow buttons until the word “TIME” is displayed. Press the TICK (✔) button to enter and edit the time settings.



[3] You should see a display with the message blinking “12hr” or “24hr”, this means you are ready to set the meter’s time. Select the 12 or 24 hour clock format with

the DOWN (▼) or UP (▲) arrow buttons. The 12 hour clock display option is indicated by a 12hr. The 24 hour clock display option is indicated by a 24hr. Your selection will be highlighted by the blinking of the symbol. Press the TICK (✔) button to confirm your selection and continue on to the hour setting.



[4] Set the hour (flashing) with the DOWN (▼) or UP (▲) arrow buttons. Press the TICK (✔) button to confirm your selection and continue on to the minute setting.



[5] Set the minute (flashing) with the DOWN (▼) or UP (▲) arrow buttons. Press the TICK (✔) button to confirm and return to the menu.



14 Setting the Date

[1] When your meter is deactivated, press the TICK (✔) button once to enter menu mode. The word “MEM” is displayed.

[2] Press the DOWN (▼) or UP (▲) arrow buttons until the word “DATE” is displayed. Press the TICK (✔) button to enter and edit the date settings.



[3] You should see a display with the message blinking “M-d” or “d-M”, this means you are ready to set the


meter's date. Select the M-d (Month-Day) or d-M (Day-Month) date format with the DOWN (▼) or UP (▲) arrow buttons. Your selection will be highlighted by the blinking of the symbol. Press the TICK (✓) button to confirm your selection and move on to the year setting.



[4] Set the year (flashing) with the DOWN (▼) or UP (▲) arrow buttons. Press the TICK (✓) button to confirm your selection and move on to the month setting.



[5] Set the month (flashing) with the DOWN (▼) or UP (▲) arrow buttons. Press the TICK (✓) button to confirm your selection.

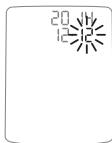

[6] Set the day (flashing) with the DOWN (▼) or UP (▲) arrow buttons. Press the TICK (✓) button to confirm your selection and return to the menu.



15 Setting Hypoglycaemic Warning Alarms

The Hypoglycaemic Warning Alarm alerts you when your glucose test results are below the hypoglycaemic threshold you have set. The meter is pre-set to a hypoglycaemic threshold of 3.9 mmol/L. Always consult your healthcare professional for your recommended target ranges. When your glucose test results are below your set hypoglycaemic threshold, the meter display will flash your test results, the abbreviation “HYPO,” time, and date repeatedly.

WARNING: Your healthcare professional should recommend what your hypoglycaemic limit is.

[1] When your meter is deactivated, press the TICK (🗑️) button once to enter menu mode. The word “MEM” is displayed.


[2] Press the DOWN (⏴) or UP (⏵) arrow buttons until the abbreviation “HYPO” is displayed.



[3] Press the TICK (🗑️) button to enter and edit the Hypoglycaemic Warning Alarm settings.



[4] Set the Hypoglycaemic Alarm ON or OFF with the DOWN (⏴) or UP (⏵) arrow buttons. Your selection will be highlighted by the blinking of the symbol. Press the TICK (🗑️) button to confirm your selection and move on to the hypoglycaemic threshold setting.

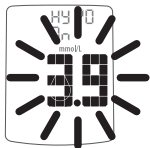

[5] Set the hypoglycaemic threshold (flashing) with the DOWN (⏴) or UP (⏵) arrow buttons. Press the TICK (🗑️) button to confirm your selection and return to the menu.

16 Setting Hyperglycaemic Warning Alarms

The Hyperglycaemic Warning Alarm alerts you when your glucose test results are above the hyperglycaemic threshold you have set. The meter is pre-set to a hyperglycaemic threshold of 13.3 mmol/L. Always consult your healthcare professional for your recommended target ranges. When your glucose test results are above your set hyperglycaemic threshold, the meter display will flash your test results, the abbreviation “HYPr,” time, and date repeatedly.

WARNING: Your healthcare professional should recommend what your hyperglycaemic limit is.

[1] When your meter is deactivated, press the TICK (🗑️) button once to enter menu mode. The word “MEM” is displayed.

[2] Press the DOWN (⏴) or UP (⏵) arrow buttons until the abbreviation “HYPER” is displayed.



[3] Press the TICK (🗑️) button to enter and edit the Hyperglycaemic Warning Alarm settings.



[4] Set the Hyperglycaemic Alarm ON or OFF with the DOWN (⏴) or UP (⏵) arrow buttons. Your selection will be highlighted by the blinking of the symbol. Press the TICK (🗑️) button to confirm your selection and move on to the hyperglycaemic threshold setting.



[5] Set the hyperglycaemic threshold (flashing) with the DOWN (⏴) or UP (⏵) arrow buttons. Press the TICK (🗑️) button to confirm your selection and return to the menu.


17 Setting The Glucose Target Range

The AgaMatrix WaveSense JAZZ includes a unique positive feedback feature where a happy face symbol is displayed with test results that fall within the glucose target range. By setting a target range for what your glucose levels should be before or after a meal, you customize when this symbol appears. Always consult your healthcare professional for your recommended target ranges.

IMPORTANT: First set the hypoglycaemic and hyperglycaemic warning alarms to your recommended thresholds before making changes within the glucose target range menu (refer to chapters 15-16 for setting these alarms).

Note: The glucose target range is limited by the values set for your hypoglycaemic and hyperglycaemic warning alarms. Your meter will not allow you to select a number beyond these alarm settings, even if the warning alarm is turned off.

For example: If you want the pre meal target range to begin at 5.0 mmol/L, then the hypoglycaemic alarm needs to be set at 5.0 mmol/L or less; or if you want the post meal target range to end at 10.0 mmol/L, then the hyperglycaemic alarm needs to be set at 10.0 mmol/L or higher.

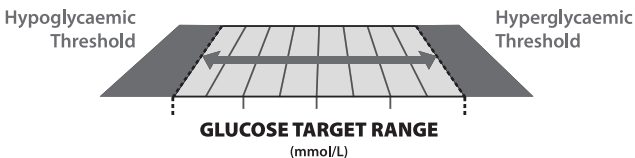

[1] When your meter is deactivated, press the TICK (🔒) button once to enter menu mode. The word “MEM” is displayed.

[2] Press the DOWN (⬇️) or UP (⬆️) arrow buttons until the word “RANGE” is displayed.




[3] Press the TICK (🔒) button to enter and edit the glucose target range settings.


[4] Set the lower limit of the pre meal glucose target range (flashing) with the DOWN (⬇️) or UP (⬆️) arrow buttons. Press the TICK (🔒) button to confirm your selection and continue on to the upper limit pre meal setting.




[5] Set the upper limit of the pre meal glucose target range (flashing) with the DOWN (⬇️) or UP (⬆️) arrow buttons. Press the TICK (🔒) button to confirm your selection and continue on to the lower limit post meal setting.

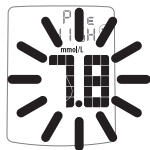

[6] Set the lower limit of the post meal glucose target range (flashing) with the DOWN (⬇️) or UP (⬆️) arrow buttons. Press the TICK (🔒) button to confirm your selection and continue on to the upper limit post meal setting.

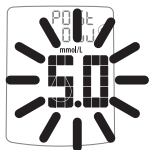

[7] Set the upper limit of the post meal glucose target range (flashing) with the DOWN (⬇️) or UP (⬆️) arrow buttons. Press the TICK (🔒) button to confirm your selection and return to the menu.

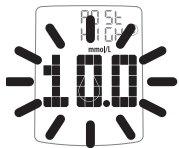

18 Setting Alarms

There are 7 alarms you can set to remind you when to test your glucose levels.

When alarms are activated:

[1] If an alarm is set and the AgaMatrix WaveSense JAZZ Meter is deactivated, the alarm will activate the meter. “AL1” thru “AL7” (depending which alarm is set) is displayed. The display backlight flashes, and the meter will beep repeatedly for 15 seconds.

[2] You can turn the alarm OFF by pressing any button.

[3] The alarms will not appear during menu and test modes. If an alarm is set to sound or flash while using these modes, it will alarm after the meter is deactivated.

CAUTION: Follow your healthcare professional’s recommendations on when and how often to test your glucose.

How to set alarms:

[1] Enter MEM Mode: When your meter is deactivated, press the TICK (✔) button once to enter menu mode. The word “MEM” is displayed.

[2] Enter Alarm Option: Press the DOWN (▼) or UP (▲) arrow buttons until the word “ALARM” is displayed. Press the TICK (✔) button to enter and set an alarm.



[3] Select An Alarm to Set: Choose the alarm you want to set with the DOWN (▼) or UP (▲) arrow buttons. If this is your first time setting an alarm, use AL1. Press the TICK (⌚) button to confirm your selection and advance to the alarm ON or OFF section.



[4] Turning Alarms ON or OFF: Turn the alarm ON or OFF with the DOWN (▼) or UP (▲) arrow buttons. Your selection will be highlighted by the blinking of the symbol. Press the TICK (⌚) button to confirm your selection and move to the set alarm time section.



[5] Set Alarm Time: You can now set the alarm time. First, set the hour (flashing numbers) with the DOWN (▼) or UP (▲) arrow buttons. Accelerated change is possible by holding down the arrow buttons. Press the TICK (⌚) button to confirm the hour and advance to the minutes.



Set the minute (flashing numbers) with the DOWN (▼) or UP (▲) arrow buttons. Press the TICK (⌚) button to confirm the minutes and return to the MENU Mode.



Note: The volume of an alarm will be affected by your meter's volume setting, which can be changed in the meter's "BEEP" option. If your meter has the volume turned OFF, then the alarm will not sound, but the backlight will flash.

[6] Setting Multiple Alarms: To set other alarms, press the DOWN (▼) or UP (▲) arrow buttons until the alarm you want to set (AL1 to AL7) is displayed. Follow the previous steps in this section.

The AgaMatrix WaveSense JAZZ Blood Glucose Monitoring System has been designed to work accurately under normal conditions. On rare occasions, the meter will display an error message instead of a glucose result. The AgaMatrix WaveSense JAZZ System is able to detect specific problems which may allow you to determine the cause of the error without wasting valuable test strips. When you are presented with an error message, check for common problems within this chapter that may explain the source of the error.

CAUTION: In certain cases, the meter may return an error code or provide an inaccurate result if it is being used near electrical equipment, like a power generator or a computer monitor. If this is the case, move the meter away from the source of electrical interference.

CAUTION: If you get repeated error messages and are experiencing symptoms of hypo or hyperglycaemia, contact your healthcare professional immediately as this may indicate low or high glucose. If this error code persists on retesting, consult your healthcare professional.

ER.1 Error 1:

Problems have occurred that are related to test strip use.

[1] If this message appears the moment the test strip is inserted, the test strip may be wet or damaged.

[2] If the message appears during a test, the test strip may

have been removed too early.

[3] If this message appears after the last progress bar instead of a result, it may indicate that you applied more blood after testing began.

ER.2 Error 2:

The meter has detected an irregularity with the sample.

[1] The test strip may be partially filled; you should check the window of the test strip to confirm that it is full.

[2] The sample may not be blood or control solution.

ER.3 Error 3:

The meter has detected that the test strip is in poor condition.

[1] The test strip may have been improperly stored (e.g. hot, humid conditions) or may be expired.

[2] The test strip may have been mishandled by vigorous bending or shaking.

[3] Parts of the test strip may have become covered in grease, oil, or lotion.

[4] The test strip port connector may be dirty.

ER.4 Error 4:

The meter was unable to produce a result or unusual test strip problems have occurred that may be related to extreme conditions.

[1] This error may be caused by extremely high glucose. If you are experiencing symptoms of hyperglycaemia, contact your healthcare professional immediately. If this error code persists

on retesting, consult your healthcare professional.

[2] Retest where it is closer to a room temperature of 21 °C to 24 °C (70 °F to 75 °F).

ER.5 Error 5:

The meter was unable to produce a reliable result.

[1] This may be caused by a non-blood sample, or a combination of high glucose and other medical conditions.

[2] If this error code persists on retesting, consult your healthcare professional.

ER.6 Error 6:

The test strip has taken too long to generate a signal.

[1] This may be caused by a combination of cold operating temperature and high haematocrit levels. Retest in a warmer location.

[2] If this error code persists on retesting, consult your healthcare professional.

ER.7 Error 7:

Meter problems have occurred that are beyond your control.

Contact Customer Care for assistance.

TEMP Temp Icon:

Test was performed outside the system operating temperature range of 10 °C to 40 °C (50 °F to 104 °F).

Move to an area with an ambient temperature of 10 °C to 40 °C (50 °F to 104 °F). Wait for the meter and test strips to reach the new temperature (usually 10-20 minutes) and retest.



Battery Icon:
Battery #2 has died during test.

Replace battery #2 immediately before retesting.

Troubleshooting Situation #1: Meter does not enter the test mode after inserting an AgaMatrix WaveSense JAZZ Test Strip.

[1] CAUSE: The batteries have insufficient power.

ACTION: Replace the batteries, set time and date, and test.

[2] CAUSE: The batteries are installed incorrectly or there are no batteries in the meter.

ACTION: Check that both batteries are installed correctly, with the positive (+) sign facing up towards you.

[3] CAUSE: AgaMatrix WaveSense JAZZ Test Strip has been inserted upside down, wrong end in, or incompletely inserted into the meter.

ACTION: Insert the AgaMatrix WaveSense JAZZ Test Strip with the black side up and the contact bars of the test strip in the meter. Ensure that the test strip is fully inserted.

[4] CAUSE: Defective meter or defective AgaMatrix WaveSense JAZZ Test Strips.

ACTION: Call customer care.

[5] CAUSE: Blood or foreign objects introduced into the AgaMatrix WaveSense JAZZ Meter test strip port.

ACTION: Please refer to chapter 22. If problem persists, call customer care and have the meter serial number available (located on back of the meter).

Troubleshooting Situation #2: The glucose test sequence does not start after applying the blood sample.

[1] CAUSE: Defective test strip.

ACTION: Repeat the test with a new AgaMatrix WaveSense JAZZ Test Strip. If problem persists, call Customer Care and have the test strip LOT number available (located on the test strip vial).

[2] CAUSE: Sample applied after meter times out and deactivates.

ACTION: Repeat the test using a new AgaMatrix WaveSense JAZZ Test Strip. Wait until you see the blood drop symbol and the word "Fill" on the display before you apply the blood sample.

[3] CAUSE: Sample not applied to the tip of the test strip.

ACTION: Repeat the test using a new AgaMatrix WaveSense JAZZ Test Strip.

[4] CAUSE: Defective meter.

ACTION: Call Customer Care and have the meter serial number available (located on the back of the meter).

CAUTION: *Low or high blood glucose test results can indicate a potentially serious medical condition. Follow your healthcare professional's recommendations.*

LO MESSAGE: Flashing message, with double beep (if volume is enabled).



MEANING: Glucose test result is lower than 1.1 mmol/L. This Lo result may indicate hypoglycaemia (low blood glucose). The Lo result is stored in the memory with time and date. It will be included in averages as a value of 1.1 mmol/L.

ACTIONS: If you feel symptoms such as weakness, sweating, nervousness, headache or confusion, follow your healthcare professional's recommendations. If you get a Lo glucose test result but have no symptoms of low blood glucose, then retest with a new AgaMatrix WaveSense JAZZ Test Strip. If you still get a Lo test result, follow the treatment plan recommended by your healthcare team or contact your healthcare professional immediately.

HI MESSAGE: Flashing message, with double beep (if volume is ON).



MEANING: Glucose test result is above 33.3 mmol/L. This Hi result may indicate hyperglycaemia (high blood glucose). The Hi result is stored in the memory with time and date. It will be included in averages as a value of 33.3 mmol/L.

ACTIONS: If you feel symptoms such as fatigue, thirst,

excessive urination, or blurry vision, follow your healthcare professional's recommendations. If you get a Hi glucose test result but have no symptoms of high blood glucose, then retest with a new AgaMatrix WaveSense JAZZ Test Strip. If you still get a Hi glucose test result, follow your healthcare professional's recommendations. Checking ketones may be advisable.

HYPOGLYCAEMIC WARNING ALARM: Flashing message with the time. Test result flashes, with double beep (if volume is ON).

HYPO

MEANING: If the Hypoglycaemic Warning Alarm is set, your glucose test result is lower than the pre-set threshold.

HYPERGLYCAEMIC WARNING ALARM: Flashing message with the time. Test result flashes, with double beep (if volume is ON).

HYP

MEANING: If the Hyperglycaemic Warning Alarm is set, your glucose test result is higher than the pre-set threshold.

KETONE MESSAGE: Flashing message, with double beep (if volume is ON).

K

MEANING: The result of the glucose test is higher than 13.3 mmol/L. **ACTIONS:** You should check ketones. Follow your healthcare professional's recommendations.

TEMPERATURE MESSAGE: Flashing message, with double beep (if volume is ON).

TEMP

MEANING: The test has been performed outside the operating temperature range. **ACTIONS:** Move to an area with an ambient temperature of 10 °C to 40 °C (50 °F to 104 °F). Wait for the meter and test strips to reach the new temperature (usually 10-20 minutes) and retest.

LOW BATTERY MESSAGE: Flashing message



MEANING: Battery #2 in your meter is low. **ACTIONS:** If the low battery symbol appears, battery #2 must be replaced. Battery numbers are indicated by the numbers printed on the pull tabs.

DEAD BATTERY MESSAGE: Flashing message



MEANING: Battery #2 in your meter is dead. **ACTIONS:** If the dead battery symbol appears, the dead battery must be replaced.

LOW BACKLIGHT BATTERY: Backlight flashes when user presses the BACKLIGHT (💡) button.

MEANING: Battery #1, which powers the backlight, is low. **ACTIONS:** If the backlight flashes when user presses the BACKLIGHT (💡) button, battery #1 must be replaced. If the backlight does not turn on, battery #1 is dead and must be replaced. (Battery numbers are indicated by the numbers printed on the pull tabs).

CAUTION: *When the battery symbol appears, you should replace the batteries immediately. Only use CR2032 or DL2032 batteries.*

Your AgaMatrix WaveSense JAZZ Meter comes with two pre-installed, CR2032, 3 volt, lithium batteries. Battery #1 powers the backlight and battery #2 powers the basic meter functions. When the power from battery #2 is low, the low battery symbol or dead battery symbol will be displayed on the screen until you change battery #2.

Two replacement CR2032, 3 volt, lithium batteries are included in the pocket of your meter's carrying case.

When replacing battery #2, the time and date information may be lost after the battery is removed from the meter. If this happens, activate the meter after installing the new battery and it will prompt you to set the time and date. Correct time and date settings are needed to get correct averages.

Removing the batteries does not affect the meter's memory or previously stored user settings, such as time format, alarms, etc.

Note: Dispose of used batteries according to your local environmental regulations. Your meter is an electronic device. When disposing of your meter follow all local environmental regulations.

Replacing the Batteries


[1] Open Battery Door: Make sure the meter is deactivated. Turn the meter so you are looking at the battery door. Press the battery door clip to open. Remove the battery door. Be careful not to misplace the battery door.



[2] Remove and Install the meter battery: To remove the used battery, pull the appropriate tab to pop out the battery. A replacement battery is included in the pocket of your meter's carrying case. Locate and install the new battery with the positive (+) sign facing up towards you.



[3] Remove and Install the backlight battery: To remove the used battery, pull the appropriate tab to pop out the battery. Install the new battery with the positive (+) sign facing up towards you.

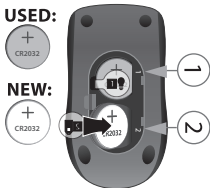

[4] Close Battery Door: Angle the battery door back onto the meter and swing down until it snaps gently into place. Make sure that the battery door is completely closed before using.



IMPORTANT: When the meter is deactivated and you are replacing battery #2, the time and date information may be lost after the battery is removed from the meter. If this happens, activate the meter after installing the new battery and it will prompt you to set the time and date.

Time and Date Setting After Battery Replacement

If the date and time need to be set, the word "SET" will be displayed the first time the meter is activated.

Note: The meter retains the date and time format [24 or 12 hour time, M-d (Month-Day) or d-M (Day-Month)]

[1] Set the year (flashing) with the DOWN (▼) or UP (▲) arrow buttons. Press the TICK (✔) button to confirm your selection.



[2] Set the month (flashing) with the DOWN (▼) or UP (▲) arrow buttons. Press the TICK (✔) button to confirm your selection.



[3] Set the day (flashing) with the DOWN (▼) or UP (▲) arrow buttons. Press the TICK (✔) button to confirm your selection.



[4] Set the hour (flashing) with the DOWN (▼) or UP (▲) arrow buttons. Press the TICK (✓) button to confirm your selection.



[5] Set the minute (flashing) with the DOWN (▼) or UP (▲) arrow buttons. Press the TICK (✓) button to confirm your selection. If the meter was activated via strip insertion, pressing TICK (✓) allows you to perform a test. If the meter was activated via the TICK (✓) button, the meter will enter menu mode.



22 Caring For Your Meter and Lancing Device

When using your meter, avoid getting dirt, dust, blood, control solution, water or any other liquid into the test strip port and battery compartment. Clean the outside of the meter or the lancing device using a cloth dampened with mild detergent or mild soap. If you wish to clean the lancing device cap only, remove the cap, wash it in warm water, rinse well, and dry.

CAUTION: Never immerse the meter or the body of the lancing device in water or any other liquid solution.

23 Meter Specifications

Assay Method: Dynamic Electrochemistry

Maximum Altitude: 3,048 meters (10,000 feet)

Calibration: Plasma equivalent

Coding: No Code

Sample: Whole blood, capillary

Sample Size: 0.5 microliters

Glucose Test Time: As fast as 5 seconds (average test time 6 seconds)

Measurement Units: mmol/L

Alarms: 7 programmable alarms

Result Range: 1.1 to 33.3 mmol/L

Haematocrit: 20% to 60%

Operating Relative Humidity: Up to 90%

Operating Temperature: 10 °C to 40 °C (50 °F to 104 °F)

Control Solution Storage Temperature: 2 °C to 30 °C (36 °F to 86 °F)

Test Strip Storage Temperature: 8 °C to 30 °C (46 °F to 86 °F)

Memory: 1865 blood glucose and control solution test results with date, time, and averaging (14, 30, 90 days and mealtimes)

Power Source: Two replaceable CR2032, 3 volt, lithium batteries

Automatic Deactivation: 3 minutes after last user action

Size: 4.6 cm x 8.3 cm x 2.0 cm (1.81" x 3.27" x 0.79")

Weight: 48 g (including batteries)

Equipment not suitable for use in the presence of flammable mixtures.

This meter complies with the applicable standards for Electromagnetic Compatibility; however it is not recommended that it be used in the presence of strong electromagnetic fields. When using this meter, keep away from sources of electromagnetic disturbances such as electric motors or radio transmitting equipment. This meter should not affect the normal operation of other devices. If this does occur, provide more distance between the meter and the affected device.

The following symbols may appear on the equipment packages or in the documents.

Serial Number: 


Manufacturer: 

In Vitro Diagnostic Medical Device: 

Batch Code: 

Consult Instructions For Use: 

Do Not Reuse: 

Temperature Limitation: 

Use By: 

Catalogue Number: 

