


HOW THE ALLY SYSTEM WORKS

The ALLY Diabetes Management System makes tracking your diabetes data easy for your Diabetes Care Team. Results sync to the cloud for secure storage, ready for review when your Healthcare Professional (HCP) wants to, using a choice of reports and graphs to help them identify trends and aid their decision making.

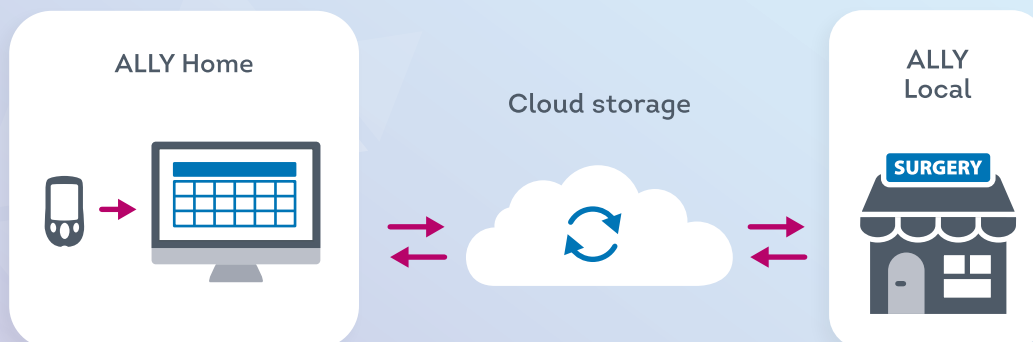

WaveSense JAZZ

The WaveSense JAZZ is compatible with the ALLY Home version of the ALLY Diabetes Management System, a free of charge PC application. Each time the patient connects their WaveSense JAZZ meter to their PC, all results are uploaded to the system in a matter of seconds.

The WaveSense JAZZ connects to a patient's home PC with a free of charge AgaMatrix cable.

The Cloud

Diabetes data automatically syncs from the ALLY Home software to the GDPR and ISO-compliant ALLY cloud. The secure storage of data in the cloud means it can be retrieved at any time by any registered user of the patient's connected Diabetes Team via ALLY Local.

The patient will need to agree to data being processed and the Privacy Policy during the registration process.

PATIENT INSTRUCTIONS FOR GETTING STARTED WITH ALLY

Getting Started

You will need the following items:



WaveSense JAZZ
blood glucose meter*



A PC



AgaMatrix cable**
(free of charge)

Connect to your Diabetes Care Team

New to ALLY?

1. You will receive a '**Welcome to ALLY email**' from your Diabetes Care Team with a link to download the software to your PC, a username (your email address) and a temporary password.
2. Click on the link.
3. Download the installation file and click on '**Launch**' to complete installation.
4. When the software installation is complete, you will be directed to the ALLY Home Login page.
5. Enter your username and the password you received in the email.
6. The system will ask you to change the temporary password to one of your choice.
7. On entering the system, a '**Connection Request**' box will appear requesting you authorise the connection between yourself and your Diabetes Care Team. Select '**Confirm**' to create the connection.
8. Follow the account opening instructions.

Already an ALLY Home account holder?

Your healthcare professional will run a search on their ALLY system and will send you a '**Connection Request**'. The next time you login into your ALLY Home account, the first thing you will see is a '**Connection Request**' box requesting you authorise the connection between yourself and your Diabetes Care Team. Select '**Confirm**' to create the connection.

Downloading Results from the WaveSense JAZZ to ALLY

1. Navigate to '**Data Management**' and select '**Download Data**'.
2. Insert the rectangular cable end into your computer's USB port.
3. Insert the round cable end into the JAZZ data port located at the top of the meter. The rubber flap can be lifted with a fingernail.
4. Choose JAZZ and click on '**Next**'.
5. You will receive a message '**Data was successfully saved**' and a green bar stating 100% download.
6. Select '**Finish**'.

* Obtain your meter from your diabetes healthcare professional.

** Obtain a free of charge cable by calling AgaMatrix Customer Care on 0800 093 1812 or by visiting www.agamatrix.co.uk to order online.