



GDm-Health™

The GDm-Health app is free to download from the Apple App store (iPhone) and Google Play (Android)

Downloading the app is easy, just follow the instructions below:

1 Navigate to the App Store or Google Play on your phone.




2 Search for, and select the **GDm-Health app**.
Download and install the app.

3 Open the app, tap **Activate**
and submit the 9-character
code your diabetes care team
supplied to you.

4 Read and agree to the **Terms of use**
by tapping on **I agree**.

If you have questions about taking blood glucose readings or using the GDm-Health app, please contact your Diabetes Specialist Midwife or care team.

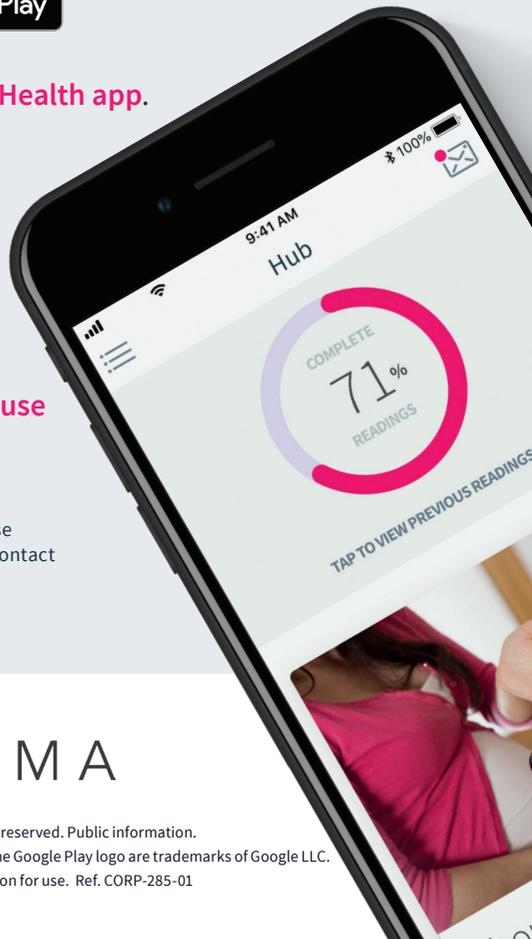

 H U M A

© 2023 Huma Therapeutics All rights reserved. Public information.

App Store® is a trademark of Apple Inc. Google Play and the Google Play logo are trademarks of Google LLC.

Subject to contract. Refer to instruction for use. Ref. CORP-285-01

Connect a WaveSense *JAZZ WIRELESS* blood glucose meter to the GDm-Health app



Pairing the AgaMatrix WaveSense *JAZZ WIRELESS* meter to your smart device is easy, just follow the instructions below:

1. Open the GDm-Health app and from the 'Hub' screen, tap 'connect to a meter'
2. Tap the WaveSense *JAZZ WIRELESS* meter in the 'Choose meter' list
3. Press and hold the button on your WaveSense *JAZZ WIRELESS* meter until the Bluetooth symbol starts flashing, then tap the 'Scan now' button
4. The meter and its serial number will appear in your list of meters, tap on it to begin pairing
5. When prompted, enter the 6-digit code displayed on your meter into the app; a confirmation should appear once the meter is successfully paired

.....

To sync blood glucose readings from the meter to the app simply:

1. Open the GDm-Health app and from the 'Hub' screen, tap the large  symbol on the bottom right of the screen
2. Tap 'Take BG reading'
3. Press and release the button on your meter and tap 'Scan' on the app to sync recent readings

7500-10223 Rev C