



The WaveSense JAZZ™ WIRELESS works in conjunction with the AgaMatrix Diabetes Manager app and allows for remote management of diabetes.

The WaveSense JAZZ™ WIRELESS automatically syncs blood glucose results to the AgaMatrix Diabetes Manager app where you can also log and track other diabetes related data such as insulin dose, carbohydrate intake and weight.

The app has an email function which enables you to send a pre-formatted report at the push of a button to your health professional. Automatic notifications can also be set up so family, friends or carers receive a real-time notification every time you test.



How to download the free of charge AgaMatrix Diabetes Manager app:

If you own an Apple device, go to the App Store.
If you own an Android device, go to Google Play.



In the search bar, start typing "**AgaMatrix Diabetes Manager**" and follow the instructions to install the app. Once download has completed, follow the in-app instructions to set up your profile.

How to pair your WaveSense JAZZ blood glucose meter to your mobile device or tablet:


1. Make sure that the Bluetooth® is active on your mobile device.
2. Open the app and navigate to the menu icon made of three lines.
3. Tap on **'Meters'**. Tap **'Add a new meter'**.
4. Press and hold down the button on the meter. You will see a flashing Bluetooth symbol.
5. The meter's serial number will appear in the list of meters within the app; Tap on it.
6. Input the 6-digit passkey that appears on your meter screen into your app and confirm by tapping on **'pair'**.
7. Both the meter and the app will indicate success. The meter will have a tick mark and the app will show a message.



If you need help in any aspect of pairing, syncing or sharing data please call AgaMatrix Customer Care on **0800 093 1812**

Syncing will occur when:

- Bluetooth on the supported device is on
- Your meter and supported device are paired
- Your meter and supported device are within 10 metres of each other
- Your meter is on

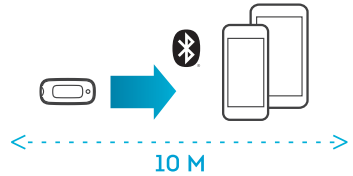



Note: Test results taken prior to the time and date being set will not have a time and date associated with them on the meter, and will not sync to the app. You can manually add these readings to the app to include them in your statistics.

How to email your results to your healthcare professional

It's important to note that to share your results in this way, you must have access to a wireless network and have email set up on your device which also has the AgaMatrix Diabetes Manager app downloaded.



Launch the AgaMatrix Diabetes Manager app on your device by tapping on the app icon on your home screen.

1. Navigate to the dashboard or timeline.
2. Tap on the '**share**' icon in the bottom left corner. The icon will look like this  on iOS devices or will look like this  on Android devices.
3. You will then be asked which time frame you want to share your data from, 7, 14, 30 or 90 days.
4. A new window will open with an email containing a pre-formatted report of your results for the chosen time frame. In the "**To**" box, write your healthcare professionals email address.
5. Before pressing send, please change the title of the email from "**AgaMatrix Diabetes Manager Report**" to your **full name and date of birth**.
6. Press send.

How to set up instant notifications to friends, family and carers

(Not to be confused with a request from your healthcare professional for you to email your results).



Launch the AgaMatrix Diabetes Manager app on your device by tapping on the app icon on your home screen.

1. Navigate to the menu icon made of three lines.
2. Tap on '**sharing**'.
3. Tap on '**Share with someone**' to add the email address of the person with whom you wish to share your glucose data. That person must also download the app and sign up for an account. Once you send the invitation, that person will receive instructions on how to download the app, accept your invitation and see your data.