


Diabetes Management System

HOW THE ALLY SYSTEM WORKS

The ALLY Diabetes Management System makes tracking your diabetes data easy for your Diabetes Care Team. Results sync to the cloud for secure storage, ready for review when your Healthcare Professional (HCP) wants to, using a choice of reports and graphs to help them identify trends and aid their decision making.

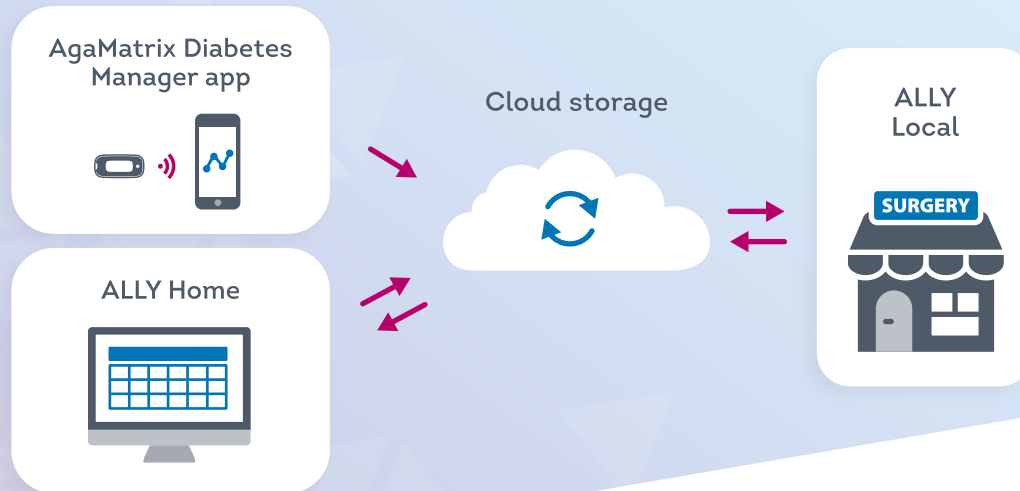

WaveSense JAZZ WIRELESS

The WaveSense JAZZ WIRELESS connects to the AgaMatrix Diabetes Manager app which gives you the ability to link to your Diabetes Team using the activation code emailed to you by your HCP.

Once that connection has been made, every time you test, your blood glucose result will seamlessly transfer to the cloud for storage ready for review by your HCP.

The Cloud

Once you've connected to your Diabetes Team, diabetes data automatically syncs from the AgaMatrix Diabetes Manager or ALLY Home application to the GDPR and ISO-compliant ALLY cloud. The secure storage of data in the cloud means it can be retrieved at any time by any registered user of the patient's connected Diabetes Team via ALLY Local.

The patient will need to agree to data being processed and the Privacy Policy during the registration process.

Diabetes Management System

PATIENT INSTRUCTIONS FOR GETTING STARTED WITH ALLY • WAVESENSE JAZZ WIRELESS

You will need the following items:



WaveSense JAZZ WIRELESS
blood glucose meter*



Smartphone

Download the AgaMatrix Diabetes Manager app to your mobile device

If you own an Apple device, go to the App Store.
If you own an Android device, go to Google Play.



In the search bar, start typing “AgaMatrix Diabetes Manager” and follow the instructions to install the app. Once download has completed, follow the in-app instructions to set up your profile.

Pair your meter to the AgaMatrix Diabetes Manager app

1. Make sure that the Bluetooth is active on your mobile device.
2. Open the app and navigate to the menu icon made of three lines.
3. Tap on ‘Meters’. Tap ‘Add a new meter’.
4. Press and hold down the button on the meter. You will see a flashing Bluetooth symbol.
5. The meter’s serial number will appear in the list of meters within the app. Tap on it.
6. Input the 6-digit passkey that appears on your meter screen into your app and confirm by tapping on ‘pair’.
7. Both the meter and the app will indicate a successful connection. The meter will have a tick mark and the app will show a message.



Connect to your Diabetes Care Team

1. Navigate to the “Menu” by tapping on the menu icon made of three lines.
2. Select “Connected Partners”.
3. Select “Add a partner”.
4. Input the activation code displayed in the “Welcome to ALLY” email you received from your Diabetes Care Team.

The connection is now made between you and your Diabetes Care Team.

Every time you test your blood glucose, your result will automatically sync with the AgaMatrix Diabetes Manager app, if your meter and mobile device are both switched on and within 10 meters of each other. The result is automatically uploaded to ALLY.

If you wish, you can download the ALLY Home application to your PC by visiting www.agamatrix.co.uk/ally and selecting “Download Now”. ALLY home enables you to view and analyse your data using a range of reports and graphs.

* Obtain your meter from your diabetes healthcare professional.