


# AgaMatrix<sup>®</sup>

 Agile™

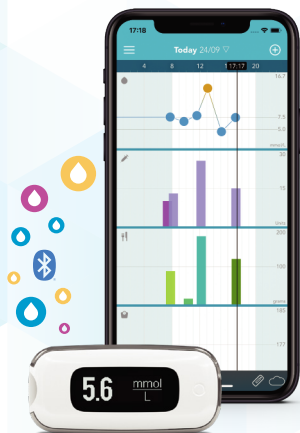

The AgaMatrix Agile™ works in conjunction with the AgaMatrix Diabetes Manager app and allows for remote management of diabetes.

The AgaMatrix Agile™ automatically syncs blood glucose results to the AgaMatrix Diabetes Manager app where you can also log and track other diabetes related data such as insulin dose, carbohydrate intake and weight.


The app has an email function which enables you to send a pre-formatted report at the push of a button to your health professional. Automatic notifications can also be set up so family, friends or carers receive a real-time notification every time you test.



## Download the AgaMatrix Diabetes Manager app to your device:

Scan the QR code with your phone's camera to download the AgaMatrix Diabetes Manager app.


## Pair your meter to the AgaMatrix Diabetes Manager app:

1. Make sure that the Bluetooth<sup>®</sup> is active on your mobile device.
2. Open the app and navigate to the menu icon. The icon will look like this  and you will find it in the top left-hand corner.
3. Tap on **'Meters'**. Tap **'Add a new meter'**.
4. Press and hold down the button on the meter. You will see a flashing Bluetooth symbol.
5. The meter's serial number will appear in the list of meters within the app; Tap on it.
6. Input the 6-digit passkey that appears on your meter screen into your app and confirm by tapping on **'pair'**.
7. Both the meter and the app will indicate success. The meter will have a tick mark and the app will show a message.

If you would like help with pairing, syncing or sharing data, please call  
AgaMatrix Customer Care on **0800 093 1812**  
or email **customer@agamatrix.co.uk**

### Syncing will occur when:

- Your meter and supported device are within 10 metres of each other
- Bluetooth on the supported device is on
- Your meter and supported device are paired
- Your meter is on

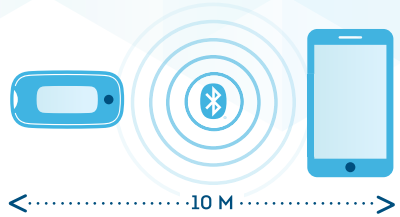


Note: Test results taken prior to the time and date being set will not have a time and date associated with them on the meter, and will not sync to the app. You can manually add these readings to the app to include them in your statistics.



## Email your results to your healthcare professional:

It's important to note that to share your results in this way, you must have access to a wireless network and have email set up on your device which also has the AgaMatrix Diabetes Manager app downloaded.



Launch the AgaMatrix Diabetes Manager app on your device by tapping on the app icon on your home screen.

You must create a member account. To create a member account, tap on the menu icon . Select **'Profile'** and under the account section, tap on **'Sign up for a free account'**.

1. Navigate to the dashboard or timeline.
2. Tap on the **'share'** icon in the bottom left corner. The icon will look like this  on iOS devices or will look like this  on Android devices.
3. You will then be asked which time frame you want to share your data from; 7, 14, 30 or 90 days.
4. A new window will open with an email containing a pre-formatted report of your results for the chosen time frame. In the **"To"** box, write your healthcare professional's email address.
5. Before pressing send, please change the title of the email from **"AgaMatrix Diabetes Manager Report"** to your **full name and date of birth**.
6. Press send.